Lab Dept: Hematology

Test Name: PREGNANCY TEST, SERUM

**General Information** 

Lab Order Codes: SEHCG

**Synonyms:** HCG; pregnancy test; human chorionic gonadotropin

**CPT Codes:** 84703 – Gonadotropin, chorionic (hCG); qualitative

**Test Includes:** Qualitative serum pregnancy test reported as positive or negative.

Logistics

**Test Indications:** Useful for the qualitative detection of human chorionic gonadotropin

(hCG) to detect pregnancy.

Lab Testing Sections: Hematology

Phone Numbers: MIN Lab: 612-813-6280

STP Lab: 651-220-6550

**Test Availability:** Daily, 24 hours

Turnaround Time: 1 hour

**Special Instructions:** N/A

Specimen

Specimen Type: Blood

Container: Red top tube

**Draw Volume:** 2 mL (Minimum: 1.5 mL) blood

Processed Volume: Minimum: 0.5 mL serum

**Collection:** Routine venipuncture

**Special Processing:** Lab staff: Centrifuge specimen, remove serum aliquot into a screw-

capped plastic vial. Store at refrigerated temperatures.

Patient Preparation: None

Sample Rejection: Unlabeled or mislabeled specimen

Interpretive

**Reference Range:** Negative (reported as positive or negative).

Specimens containing as low a level as 25 mIU/mL hCG will yield positive results when tested with Quick Vue One-Step hCG Combo Test®. In normal pregnancy, hCG can be detected as early as 6 days following conception with concentrations doubling every 32 to 48 hours, peaking in excess of 100,000 mIU/mL in approximately 10 to 12 weeks. Levels of 25 mIU/mL hCG are reportedly present in urine and serum as early as 2 to 3 days before expected menses. Serum hCG is rapidly cleared into the urine and the concentration of hCG in serum is

approximately equal to the concentration in urine.

Critical Values: N/A

**Limitations:** Some instances of a positive urine and negative serum test occur. This

is due to hCG breakdown products present in the urine. Repeat the test

a week later.

Methodology: Monoclonal Antibody, Quick Vue One-Step hCG Combo Test®

**Contraindications:** This test should not be used for tumor markers, order HCG, Beta

subunit.

References: Quidel Quick Vue One-Step hCG Combo Test Kit Package Insert

(2002)