





VersaFlo PAPR Hoods

Standard Hood (Item #9786)	Double Bib Hood with Reusable Head Suspension (Item #23354)
	
<p><u>When to Wear:</u> Possible exposure to airborne infectious agents (i.e. TB)</p>	<p><u>When to Wear:</u></p> <ul style="list-style-type: none"> • Surgery cases • Employees with facial hair that interferes with face seal on standard hood - Contact Materials Management to get this item
<p><u>Before Use:</u></p> <ul style="list-style-type: none"> • Check PAPR battery for sufficient charge • Check air flow by placing air flow indicator on motor – ball should hover above line “F” 	<p><u>Before Use:</u></p> <ul style="list-style-type: none"> • Check PAPR battery for sufficient charge • Check air flow by placing air flow indicator on motor – ball should hover above line “F”
<p><u>After Use:</u></p> <ol style="list-style-type: none"> 1. Remove hose from hood (squeeze blue strip, pull apart). Place hose on cart with PAPR motor <div style="text-align: center;">  </div> <ol style="list-style-type: none"> 2. Discard entire hood appropriately after shift <p><i>NOTE: PAPR belt extenders can be obtained through Materials Management</i></p>	<p><u>After Use:</u></p> <ol style="list-style-type: none"> 1. Remove hose and coupling from hood (squeeze blue strip, pull apart). Place hose and coupling on cart with PAPR motor <div style="text-align: center;">  </div> <ol style="list-style-type: none"> 2. Remove reusable head suspension and place on cart with PAPR motor – do not throw this away! 3. Discard double bib hood appropriately after shift <p><i>NOTE: the coupling is an optional accessory. The regular hose used on a standard hood can be used on a double bib hood</i></p>
Important Contact Information	
Respiratory Therapy for Just-In-Time Training STP: x6-8480 MPLS: x5-8250	Materials Management for additional equipment requests, pick up/reprocessing STP: x6-6982 MPLS: x5-8215

The standard reprocessing guidelines apply to motor, hose, and head suspension

VersaFlo PAPR System Carts

Standard PAPR Cart Checklist

- 2 PAPR Blower motors w/hose (inside protective bag)
- 2 Batteries on multi-unit charger
- 4 Standard 433L Hoods (inside protective bags)
- 2 air flow monitors

*Contact Materials Management if double bib hood with reusable head gear is needed

Surgery PAPR Cart Checklist

- 4 PAPR Blower motors (inside protective bag) – hoses not connected to motors, rather hoods
- 4 Batteries on multi-unit charger
- 4 Hoods with reusable head gear w/hose and coupling attached (inside protective bags)
- 2 air flow monitors