



101M2

MODIFIED LAY DOWN VEST

IMPORTANT INFO

- Use for body casts, hip spica casts, braces and any other configuration where the passenger must be transported laying down.
- 101M2 for passengers weighing 40 – 106 lbs.
- Sizing according to the chest measurement 101M2 25” – 36”
- A layer of padding no thicker than 2” may be used under the passenger for added comfort.
- Can be secured by using lap type seat belts or shoulder/lap belts. No mount is needed.
- EZ-ON Products must be worn under winter jackets.
- REGISTER this transportation vest to be notified of a safety recall.

WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.

WARNING

Death or serious injury can occur if manufacturer’s instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.


PINCH TEST PROCEDURE

THE PINCH TEST

Tighten vest so that no pinch in the webbing can be achieved on the chest strap.
When vest is snug fingers will be unable to grab any webbing.



INCORRECT

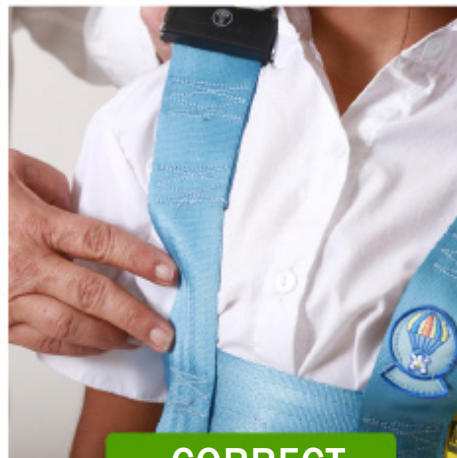

CORRECT



101M2

MODIFIED LAY DOWN VEST

INSTALLATION INSTRUCTIONS

TO SECURE VEST ONTO PASSENGER

1. Place vest around passenger with EZ-ON parachute logo on the front.
2. Close and secure push buttons.
3. Bring crotch straps through passenger's legs and secure.
4. Pull downward on strap to adjust. Pull webbing snug but not too tight. Make sure to position the hip strap low, and over hip bones.
5. Pull loose end of shoulder strap through bar slide.
6. Hold bar slide with one hand while pulling loose end to lengthen or shorten strap.
**Do not pull too tight, as this will cause vest to pull off hips.*
7. Back thread to lock in place.

PINCH TEST

Please refer to procedure on reverse side.

TO SECURE PASSENGER TO REAR SEAT

NOTE: Rear car seat must be flat. Passenger must be able to fit on rear seat laying lengthwise.

8. Lay passenger in the center of the rear seat with the casted leg against the seat back facing away from the vehicle doors.
9. For children over 30" tall route the vehicle seatbelt closest to the passenger's head through the chest strap, buckle and tighten.
**For passenger under 30" tall route seatbelt through the chest and hip strap, buckle and tighten.*
10. Route the second seat belt through the hip strap, buckle and tighten.
11. Secure seat belts as tightly as possible. Make sure all the seat belt are in the locked position.
12. Place pillows behind passenger's head and neck for comfort and protection.

USE OF EXTREMITY BELT

If 1 leg is casted, be sure that the cast is placed against the rear seat. If both legs are cast, place the extremity belt around both casts at the knee area.

13. Place extremity belt around casted leg(s) and the lap portion of the already buckled seatbelt.
14. Position the extremity belt at or above the child's knee, buckle the extremity belt and pull to tighten.
15. Secure extremity belt by pulling it snug.





M203

MODIFIED LAY DOWN VEST

IMPORTANT INFO

- Use for body casts, hip spica casts, braces and any other configuration where the passenger must be transported laying down.
- M203 for passengers over 12 months old, weighing 22 – 75 lbs.
- Sizing according to the chest measurement M203 18” – 24” | 101M2 25” – 36”
- A layer of padding no thicker than 2” may be used under the passenger for added comfort.
- Can be secured by using lap type seat belts or shoulder/lap belts. No mount is needed.
- EZ-ON Products must be work under winter jackets.
- REGISTER this transportation vest to be notified of a safety recall.

WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.


WARNING

Death or serious injury can occur If manufacturer’s instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

PINCH TEST PROCEDURE

THE PINCH TEST

Tighten vest so that no pinch in the webbing can be achieved on the chest strap.
When vest is snug fingers will be unable to grab any webbing.

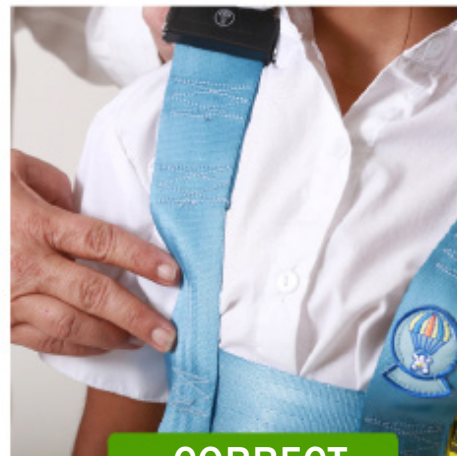



M203

MODIFIED LAY DOWN VEST



INSTALLATION INSTRUCTIONS

TO SECURE VEST ONTO PASSENGER

1. Place vest around passenger with EZ-ON parachute logo on the front.
2. Close and secure push buttons.
3. Bring crotch straps through passenger's legs and secure.
4. Pull downward on strap to adjust. Pull webbing snug but not too tight. Make sure to position the hip strap low, and over hip bones.
5. Pull loose end of shoulder strap through bar slide.
6. Hold bar slide with one hand while pulling loose end to lengthen or shorten strap.
**Do not pull too tight, as this will cause vest to pull off hips.*
7. Back thread to lock in place.

PINCH TEST

Please refer to procedure on reverse side.

TO SECURE PASSENGER TO REAR SEAT

NOTE: Rear car seat must be flat. Passenger must be able to fit on rear seat laying lengthwise.

8. Lay passenger in the center of the rear seat with the casted leg against the seat back facing away from the vehicle doors.
9. For children over 30" tall route the vehicle seatbelt closest to the passenger's head through the chest strap, buckle and tighten.
**For passenger under 30" tall route seatbelt through the chest and hip strap, buckle and tighten.*
10. Route the second seat belt through the hip strap, buckle and tighten.
11. Secure seat belts as tightly as possible. Make sure all the seat belts are in a locked position.
12. Place pillows behind passenger's head and neck for comfort and protection.

USE OF EXTREMITY BELT

If 1 leg is casted, be sure that the cast is placed against the rear seat. If both legs are cast, place the extremity belt around both casts at the knee area.

13. Place extremity belt around casted leg(s) and the lap portion of the already buckled seatbelt.
14. Position the extremity belt at or above the child's knee, buckle the extremity belt and pull to tighten.
15. Secure extremity belt by pulling it snug.